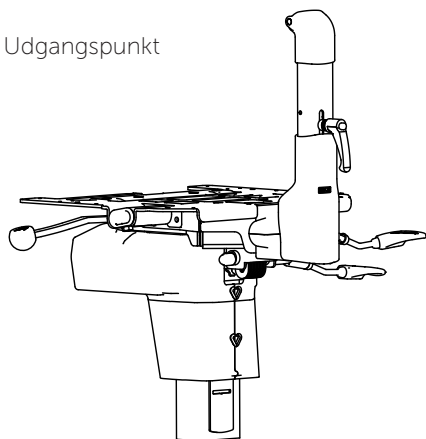
Elryg / El. back / Motor Rygg - VELA Tango 700

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ENGLISH

1. Remove the backrest from the chair by unscrewing the shift lever (A) on the chair completely off, lift off the backrest (B) and set it aside.
2. Unscrew the two M8 screws (C) that is holding the backrest stem (D) and remove it.
3. Remove the covers (E) by removing the o-rings (F) and pulling to the side. If necessary use a small screwdriver. This gives access to the batteries.
4. Remove the two screws (G) and cable relief (H) on the back of the chair with a 4 mm hex key.
5. Mount the motor control bracket (I) with the same two screws.
6. Install the motor control (J) and connect the power to the front of the batteries. Pay attention to the positive and negative terminals.
7. Reassemble the covers (E) with the o-rings (F).
8. Remove the 3 screws (K) that secure the back. They are well hidden behind the back from below. Remove the back unit (L).
9. Install the new electric back bracket (M) in the same 3 screws. Assemble the rest of the backrest.
10. Remove the two screws (N) from the side of the seat plate.
11. Mount the brackets over two bushings with the corresponding countersunk screws (O).
12. Attach the actuator (P) as shown with a shoulder screw (Q) and nut at each end. Do not tighten the nut completely.
13. Connect the wires between the actuator, the motor control and the handbox.

Important: +/- poles must be correctly oriented. The motor control is damaged if it is installed inverted.